

How to fill in the water			
Step	Action	Photo	Note
1	Insert the funnel into the "Inlet."		NOTICE Water Quality: Use only distilled or purified water. Never use corrosive liquids. Volume: Approximately 1.5 liters.
2	Insert the other water hose into the "overflow."		
3	When water begins to flow out of the overflow port, press the up arrow on the screen to pump water into the bladder and handpiece cable.		
4	Refill with water until it discharges from the overflow port again. The filling process is now complete.		

How to drain the water			
Step	Action	Photo	Note
1	Insert the hose without the funnel into the outlet port to drain water from the tank.		
2	Press the down button on the screen to drain the water from the bladder and tubing back into the water tank.		




Image of Protective Plug

→



CAUTION

PLEASE REMOVE THE PLUG from the "Inlet" port BEFORE use.

The plug is installed to prevent water leakage during shipping. Operating the device with the plug inserted will cause water tank damage.