

 **LifeEar**
Read before use.
v.A1215


 **LifeEar**



Simple Device. Enhanced Life.

Box Contents

LifeEar with tubing
& tip attached.

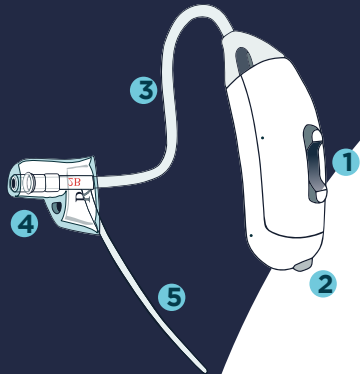

Power Tip
Large



- 1 Control Switch
- 2 Battery Door
- 3 Tubing
- 4 Standard LifeEar Tip
- 5 Ear Lock

Size 10 Batteries
(Yellow Tab)



Cleaning
Brush & Wire



Power:

ON = battery door closed
OFF = battery door open

Battery Type:


LifeEar uses a size 10 zinc-air
battery (yellow tab).

*Wait one FULL minute
after removing tab for
battery to fully charge.*

Low Battery Warning:

Three rapid double beeps


Inserting Battery

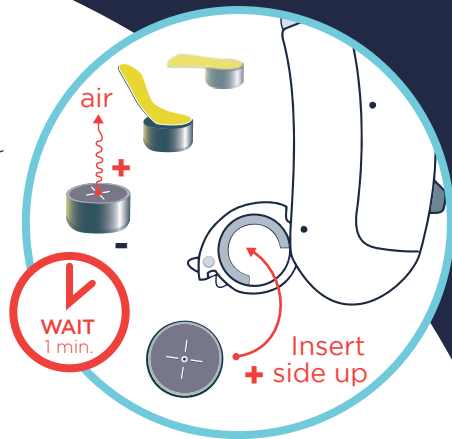


Wearing Your LifeEar

Place tip in entrance of ear canal.

1

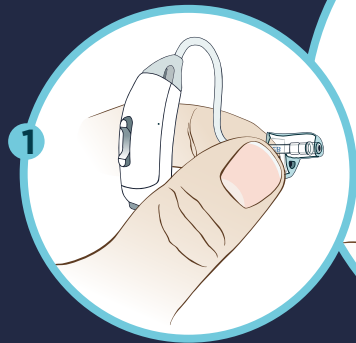


2

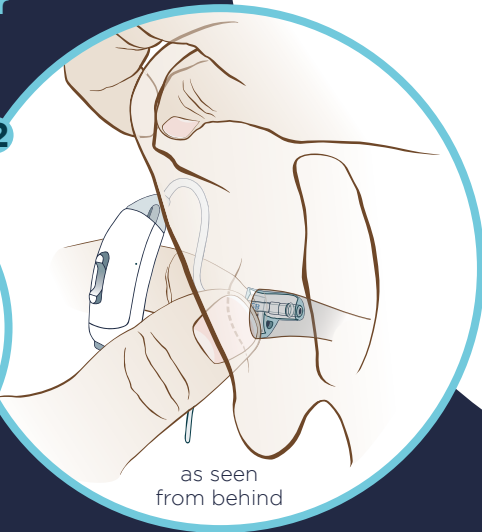


3

Place LifeEar behind ear.

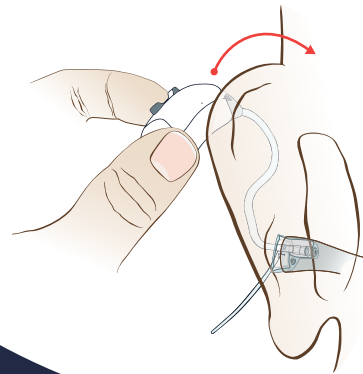


4

Move ear lock into bowl of ear.

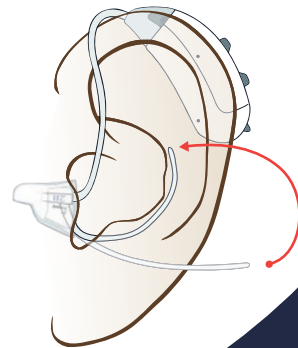

Controlling Program & Volume

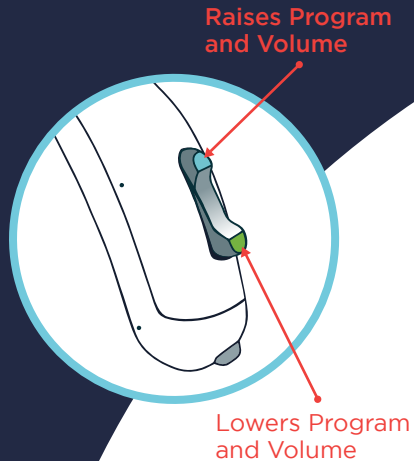

Volume: Press/Release before LifeEar beeps. Single beep when volume adjusted. Triple beeps at volume limit.

Program: Press/Hold until LifeEar beeps.

- beep ♪ = Program 1
- beep, beep ♪ ♪ = Program 2
- ♪ ♪ ♪ = Program 3
- ♪ ♪ ♪ ♪ = Program 4

Adjusting Volume:

Press and Release switch until normal conversation is comfortable.



Set volume in quiet room.


Program Settings:

- 1 Enhanced sound across all frequencies.
- 2 High-frequencies emphasized.
- 3 Program 1 plus noise reduction boost.
- 4 Program 2 plus noise reduction boost.

Replacing LifeEar Tubing & Tips





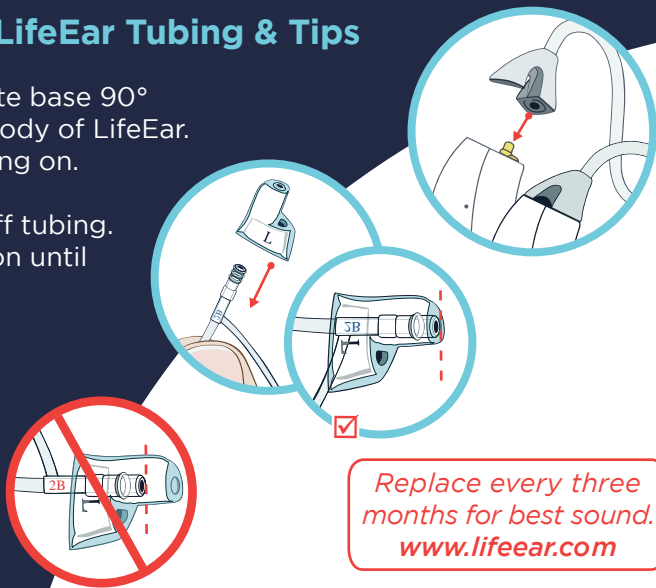

Tubing: Rotate base 90° and slide off body of LifeEar. Push new tubing on.

Tip: Pull tip off tubing. Push new tip on until flush with end of tubing.



*Replace every three months for best sound.
www.lifeear.com*

Fitting Your LifeEar


	Average Ear Canal	Large Ear Canal
Natural Sound 	included  LifeEar Tip Standard	 * LifeEar Tip Large
Maximum Output 	 * Power Tip Standard	included  Power Tip Large

*Available at www.lifeear.com

Cleaning & Care

Avoid exposure to moisture.

Clean tubing regularly with enclosed tool.

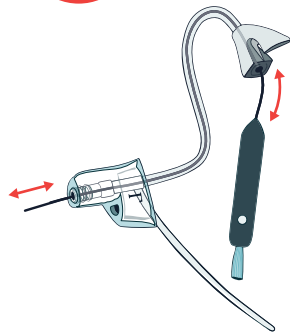

Support

www.lifear.com

Troubleshooting

Problems? Issues? Questions?
Visit www.lifear.com

Note: If your LifeEar provides no amplification with tubing attached, clean tubing thoroughly.